

EN

Instructions for Use


Ortho Chain

WARNING

1. Single use only.
2. Avoid use for patients with an identified allergy to polyurethane, such as skin sensitization effect.

CONFIGURATION AND STRUCTURE

Component: Polyurethane

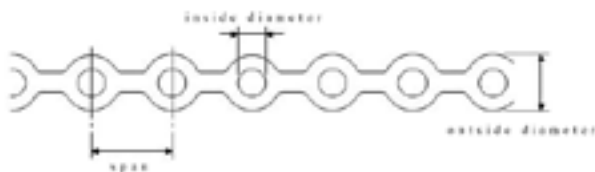

Dimensions

	Thickness	Outside diameter	Inside diameter	Span
Short span	0.2mm	2.8mm	1.4mm	3.0mm
	0.3mm			
	0.4mm			
	0.6mm			
Medium span	0.2mm	2.8mm	1.4mm	3.6mm
	0.3mm			
	0.4mm			
	0.6mm			

PERFORMANCE, INTENDED PURPOSE, EFFECT OR EFFICACY

This product is elastic material used for movement of teeth and retention of orthodontic appliances in orthodontic treatment.

OPERATING INSTRUCTIONS OR HOW TO USE

Cut into an appropriate length, tying to the tie wing of bracket or hooking on the hook, the traction force is applied.

PRECAUTIONS CONCERNING THE USAGE

1. In order to avoid the risk of infection, use as cutting into an appropriate length beforehand.
2. For traction, use with elongation of 2 -2.5 times the original for the standard, and do not apply too strong traction force to the teeth.

PRECAUTIONS FOR USE

Important basic cautions

1. Do not use the product for other than the orthodontic purposes.
2. Professional use only.
3. If deformation or deterioration is recognized, do not use. Contact a sales representative.
4. If an allergic reaction occurs, discontinue use of the product and direct patient to consult physician

Defect and adverse event

Used product is likely to cause deformation, deterioration or reduction of strength and induce infection.

STORAGE

[Storage method]

1. Keep away from non-professionals.
2. Store in a dark place, away from the direct sunshine, UV rays, heat and moisture.

[Shelf life/expiration date for use]

Please use by expiration date listed on the label.

*Expiration date uses the format YYMMDD.



GC Orthodontics Europe GmbH

Harkortstraße 2
58339 Breckerfeld
Germany

CE 2797

95-8&EI-0000 · REV02-11.2019



Since 1921
Towards Century of Health

GC Orthodontics Europe GmbH
Harkortstraße 2 · 58339 Breckerfeld · Germany
Tel +49.2338.801.888 · info.gco.germany@gc.dental